



ACS-processing - automated and sterile

1.



- Collect venous blood aseptically with the Orthogen®Device vet
- Label syringe with patient data

2.



- Incubate the Orthogen®Device vet with the plunger facing up at 37°C for 3-6 hours

3.



- Twist off drawing aid
 - Tear off pulling tab
- IMPORTANT:** Do not push or pull white vial holder

4.



- Load centrifuge symmetrically (if necessary with counterbalance)
- Centrifuge Orthogen®Device vet for 3 min. at 1500 g (RCF)

5.



- Pull the white vial holder out of the Orthogen®Device vet
- Remove the glass vial from the vial holder

6.



- Label the vial with patient details
- Use ACS or freeze at -18°C (***)
- When freezing use Steri-Tamp